BAFRY-ES

ENVIRONMENT SOLUTIONS

info@bafry.net

BAFRY ENVIRONMENTAL SOLUTION

Medical Field Catalogue

ADVANCED DISINFECTION & HYGIENE SOLUTIONS

German Technology | Scientifically Driven | Globally Certified

ABOUT US

A leading Egypt-German company specializing in hygiene and disinfectant solutions.

Committed to cleanliness, safety, and innovation, we offer high-quality products designed to protect personal and public health.

Since 1997, **BAFRY** GmbH has established itself as a trusted authority in environmental solutions.

Our range of disinfectants and advanced hygiene solutions is designed to the highest international quality standards, ensuring effective protection against harmful bacteria, viruses, and germs.

WHO ARE WE ?

OUR VISION

A leader in environmental solutions, we offer a wide range of products designed to meet the needs of unique applications, leaving a lasting impact on a cleaner, safer environment.

OUR MISSION

To create a microbe-free world and raise environmental health standards through innovative solutions tailored to meet diverse environmental challenges.

OUR COMMITMENT

- Commitment to continuous scientific research, verification, and improvement to ensure long-term reliability for customers and operating environments.
- Maintain the highest standards of quality, safety, and scientific integrity across all operations
- Our disinfectants are food grade, long-acting, broad-spectrum, with long-term efficacy against spores, fungi, bacteria, viruses, algae, and biofilm.
- Eco-Friendly, biodegradable products, with no harm to the environment.

DISINFECTANTS

1) ANTIKEIME

PRODUCT DESCRIPTION

- New generation multipurpose disinfectant, hydrogen peroxide & silver ions.
- Long acting broad spectrum against all microorganisms: bacteria, fungi, viruses, algae, spores and biofilms.
- Nontoxic, non-carcinogenic, non-mutagenic and No need to rinse after use.
- A powerful medical disinfectant for instruments, equipment, units, surfaces and floors in surgery rooms, neonatal incubators, and all clinics, especially dental and gynecology clinics and their contents. It eliminates pathogens and maintains public health in environments requiring strict contamination and infection control.

USAGE INSTRUCTION

- After washing the instruments, soak them in an (ANTIKEIME) disinfectant solution (10-30 ml/liter of water) , contact time (5-10) minutes. Then keep it on cellulose paper without rinsing until dry. Finally, prepare them and transfer them to the sterilizer.
- After washing the endoscopes, soak them in an (ANTIKEIME) solution (10-30 ml/liter of water) , contact time (5-10) minutes , then keep it on cellulose paper without rinsing until completely dry.
- Disinfect surfaces, equipment and dental units with (ANTIKEIME) solution (10-30 ml/liter of water) , contact time (1-10) minutes
- The suction lines of dental chair are disinfected using (ANTIKEIME) solution (10–30 ml/liter of water). The solution is sucked through both the high and low suction connections. A portion of the solution is also placed in the unit's spittoon. contact time 30 minutes, then the solution is drained from the suction lines. No rinsing is required.
- The water in the dental chair's reserve tank is disinfected using (ANTIKEIME) solution at a concentration of (0.5 ml/liter of water).
- The water in the facility's main feed tank is disinfected using (ANTIKEIME) solution at a concentration of (60–300 ml/cubic meter of water).

2) D-CLOUD

PRODUCT DESCRIPTION

- New innovation nanotechnology disinfectant, Quaternary ammonium compound & Branched alcohol.
- Strong against bacteria, fungi, viruses, algae and spores.
- nontoxic, non-carcinogenic and non-mutagenic and No need to rinse after use.
- A powerful medical disinfectant for instruments, equipment, units, surfaces and floors in surgery rooms, neonatal incubators, and all clinics, especially dental and gynecology clinics and their contents. It eliminates pathogens and maintains public health in environments requiring strict contamination control and infection control.

USAGE INSTRUCTION

- After washing the instruments, soak them in an (D-CLOUD) disinfectant solution (10-30 ml/liter of water), contact time (5-10) minutes. then rinsed well and keep it on cellulose paper until they are completely dry. Finally, prepare them and transfer them to the sterilizer.
- After washing the endoscopes, soak them in an (D-CLOUD) solution (10-30 ml/liter of water), contact time (5-10) minutes , then rinsed well and keep it on cellulose paper until they are completely dry.
- The suction lines of dental chair are disinfected using (D-CLOUD) solution (10–30 ml/liter of water). The solution is sucked through both the high and low suction connections. A portion of the solution is also placed in the unit's spittoon. contact time (5-10) minutes , then the solution is drained from the suction lines. then rinsed well.

PRE-DISINFECTANTS

PD-L

PRODUCT DESCRIPTION

- Alkaline pre-disinfectant that prevents infection transmission, provides effective cleaning, and prevents blood clotting and drying of medical instruments during transport and cleaning. It is also effective with human secretions such as carbohydrates, mucus, lipids, and proteins.

USAGE INSTRUCTION

- After the instruments are used, they are placed in a (PD-L) solution with a concentration of (2%) (20 ml solution / 980 ml water) to prevent the instruments from drying out and the blood clotting, then they are left in the solution during their transfer from operating rooms and clinics to the sterilization room to prevent the spread of infection during transfer.
- Keep the instruments in the solution, contact time 10 minutes and cleaned with a brush to remove any debris. then rinsed well.
- The suction lines are cleaned using (PD-L) solution at a concentration of (3%) (30 ml/970 ml water). The solution is sucked through both the high and low suction connections. A portion of the solution is also placed in the unit's spittoon. contact time 10 minutes, then rinsed well.

REMOVERS

SMC

PRODUCT DESCRIPTION

- A strong several complex substances antirust, corrosion inhibitor
- Specialized in removing rust and scales, to maintain the shine of tools and protect them from corrosion.

USAGE INSTRUCTION

- Diluted (5%) (50 ml/950 ml water).
- Keep tools from which rust and salts are to be removed for a contact time 30 minutes.
- Clean the tools with a brush.
- Rinse the tools thoroughly with water and place them on cellulose paper to dry completely.

HAND CARE

1) HAND SANITIZER LIQUID

PRODUCT DESCRIPTION

- A powerful, ready to use hand Sanitizer liquid with two types of sanitizers & skin moisturizer.
- Effective, broad spectrum against all kinds of microbes, fungi's, bacteria, viruses.
- A major hand sanitizer liquid for maintain personal hygiene and Preserves from infection and contagion.

USAGE INSTRUCTION

- Dispense sufficient amount (about 2 ml) into your hands.
- Rub your hands together, cover all surfaces of your hands (palms, back of hands, between your fingers, under nails).
- Let sanitizer to dry completely, don't wipe or rinse your hands.

2) HAND SANITIZER GEL

PRODUCT DESCRIPTION

- A powerful, ready to use hand Sanitizer gel with two types of sanitizers & skin moisturizer.
- Effective, broad spectrum against all kinds of microbes, fungi's, bacteria, viruses.
- A major hand sanitizer gel for maintain personal hygiene and Preserves from infection and contagion.

USAGE INSTRUCTION

- Dispense sufficient amount (about 2 ml) into your hands.
- Rub your hands together, cover all surfaces of your hands (palms, back of hands, between your fingers, under nails).
- Let sanitizer to dry completely, don't wipe or rinse your hands.

3) ANTIBACTERIAL HAND SOAP

PRODUCT DESCRIPTION

- A ready to use hand soap for cleaning, sanitation and moisturizing, Gentle on the skin.
- Effective cleaner & sanitizer against all kinds of microbes, fungi's, bacteria, viruses. Preserves from infection and contagion.
- Keep hands clean, protected for a long period at least 24 hours.

USAGE INSTRUCTION

- Wet your hands with clean & running water (warm or cold).
- Dispense sufficient amount (about 2 ml) into your hands, rub your hands together, cover all surfaces of your hands (palms, back of hands, between your fingers, under nails).
- Rinse your hands with water, dry your hands with a clean towel or air dryer.

4) HAND SOAP

PRODUCT DESCRIPTION

- A ready to use hand soap for cleaning, moisturizing Gentle on the skin.
- Effective cleansing soap for removing strong dirtiness (oils, grease, fats, proteins).
- Keep hands clean, protected for a long period at least 24 hours.

USAGE INSTRUCTION

- Wet your hands with clean & running water (warm or cold).
- Dispense sufficient amount (about 2 ml) into your hands, rub your hands together, cover all surfaces of your hands (palms, back of hands, between your fingers, under nails).
- Rinse your hands with water, dry your hands with a clean towel or air dryer.

5) HAND SOAP (MULTI- PERFUME)

PRODUCT DESCRIPTION

- A ready to use hand soap multi-perfume for cleaning and moisturizing.
- Effective cleansing soap for removing strong dirtiness (oils, grease, fats, proteins) that keep hands with a distinctive and refreshing fragrance that lasts for a long time to eliminate un pleasant smells.

USAGE INSTRUCTION

- Wet your hands with clean & running water (warm or cold).
- Dispense sufficient amount (about 2 ml) into your hands, rub your hands together, cover all surfaces of your hands (palms, back of hands, between your fingers, under nails).
- Rinse your hands with water, dry your hands with a clean towel or air dryer.

6) HAND FOAM (MULTI- PERFUME)

PRODUCT DESCRIPTION

- A ready to use hand foam with multi-perfume for cleaning and moisturizing.
- Effective cleansing foam for removing strong dirtiness (oils, grease, fats, proteins) that keep hands with a distinctive and refreshing fragrance that lasts for a long time to eliminate unpleasant smells.

USAGE INSTRUCTION

- Wet your hands with clean & running water (warm or cold).
- Dispense sufficient amount into your hands.
- Rub your hands together, cover all surfaces of your hands (palms, back of hands, between your fingers, under nails).
- Rinse your hands with water, dry your hands with a clean towel or air dryer.

DETERGENTS

1) DT-N PLUS

PRODUCT DESCRIPTION

- A Strong neutral cleaner and disinfectant, low foaming detergent, multi-scented. Contains a safe, organic alkaline and is phosphate-free, Suitable for all floors, walls and toilets.
- Perfect protection from germs, microbes and bacteria.
- Leaves floors shiny and protected.
- Long-lasting fragrance.

USAGE INSTRUCTION

- For daily cleaning of floors and walls, add one part of detergent to every 30 parts water.
- For bathrooms, sinks, and drains, add one part detergent to 20 parts water.

2) DT-C

PRODUCT DESCRIPTION

- A Strong acidic cleaner, medium foaming, contains a safe, organic acid and is phosphate-free, Suitable for all floors and walls.
- A Major cleaner to remove salt deposits, scales on floors and walls.

USAGE INSTRUCTION

- Diluted 30 – 50 ml / liter of water.
- Leave on the surface to be cleaned for at least 10 minutes.
- Scrub the surface using appropriate cleaning tools or high-pressure water.
- Rinse the surface thoroughly with water.

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Nr: N-58429

Federal Institute For Occupational Safety and Health

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Nr: 4638456

Federal Office For Risk Assessment

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ENVIRONMENT SOLUTIONS

CONTACT US

For technical consultation, regulatory documentation, or partnership inquiries, Bafry's regional teams are available to provide professional support aligned with your operational requirements.

MAIN OFFICE

59 Emtedad Ramsis 2, Front of Conference Center, Nasr City, Cairo, Egypt

SCIENTIFIC OFFICE

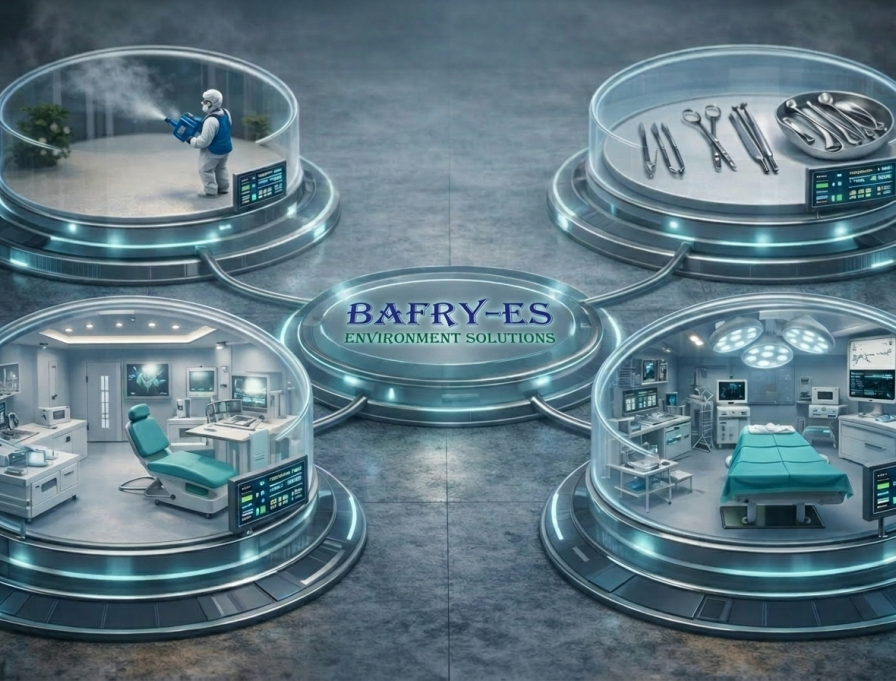
Bafry Products Company GmbH
Waldstraße 9
24960 Glücksburg - Germany

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